

# 3 - Setting Up the CryoAblation Console for a Procedure

Use the steps in this chapter to prepare the CryoAblation console for a procedure.

<b>Caution 3</b>	Ensure that equipment used with the CryoCath cardiac CryoAblation console and catheters is electrically safe and does not expose the patient to hazards.
------------------	--

<b>Caution 4</b>	Use only the catheters, accessories and refrigerant tanks that have been obtained from CryoCath Technologies Inc. with the Cryocath cardiac CryoAblation console.
------------------	---

<b>Caution 9</b>	Only appropriately trained personnel in a fully equipped medical laboratory should perform cardiac CryoAblation procedures.
------------------	---

**These steps must be performed at least 15 minutes before a procedure begins.**

## 1. Position the CryoAblation console and lock its wheels.



- Position the CryoAblation console in the desired location in the room.
- Lock the wheels of the console by pushing the levers over the front wheels down.

## 2. Open the refrigerant tank.




- Open the door on the left side of the CryoAblation console to expose the refrigerant tank.



- Open the tank by turning the knob on the top of the tank counter-clockwise.
- Close the door on the left side of the CryoAblation console.

## 3. Attach the scavenging hose, foot switch and power cable.

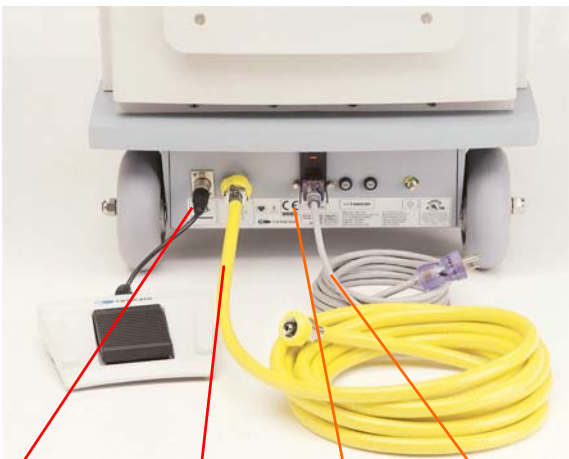

- Foot switch cable (in Europe only)
- Scavenging hose
- Wire fits over plug, holding it in position
- Power cable

- Attach one end of the yellow scavenging hose to the CryoAblation console. The connector is on the back of the console near the floor. Screw it on until it is finger tight.
- If the optional foot switch (available in Europe only) will be used during the procedure to start and stop injections, plug its cable into the connector on the back of the console near the floor.
- Attach the other end of the yellow scavenging hose to the hospital scavenging system.

(You may need to use an adaptor, depending on the type of wall connector at your hospital.)

- Attach the power cable to the outlet on the back of the console if it is not already attached. Press down on the wire located above the outlet so that it fits over the plug, holding it securely in position.

<b>Caution 17</b>	When setting up the foot switch (available only in Europe), make sure to place it in a location where it will not be inadvertently stepped on.
-----------------------	--

#### 4. Power on the CryoAblation console.



Console power switch

- Plug the CryoAblation console power cable into an outlet.
- Turn on the power switch located at the back of the console.



Touch screen power switch

- If the touch screen remains black but the console appears to be powered on, turn the power switch on the back of the touch screen off and on.



The following message appears:  
*System self test - Please Wait...*

This self test verifies that all functions and safety mechanisms are working.

If an error message appears during this test, refer to the *Troubleshooting* chapter of this manual.

The screen on the left appears when the CryoAblation console has finished powering up.